# LÜMME

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Cosmos Far View International Limited

Room 701, 16 apt, Lane 165, Rainbow North Street, Ningbo, China

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## GBR USER MANUAL

### IMPORTANT SAFETY INSTRUCTIONS

Please read this instruction carefully in order to avoid damage due to incorrect use and keep it in a safe place for future references.

- Before the first using check that permissible voltage of the appliance corresponds to the voltage of your home electrical supply network.
- Incorrect handling can cause damage to the device and cause damage to the user.
- For home use only. Do not use for industrial purposes.
- For indoor use only.
- Always unplug the appliance from the power supply when not in use.
- Close supervision is necessary when it is used near children.
- Do not leave the appliance switched on when you do not use it.
- Never carry or suspend the appliance by the cord.

- Do not operate after malfunction or damage of the cord or electric plug and after the fall of the appliance.
- Do not attempt to repair, adjust or replace parts in the appliance. Repair the malfunctioning appliance in the nearest service center.
- Do not use the appliance if you have any physical or mental disability and have no any operational experience with similar technics. In such cases the user must have the operating instructions from the person which in charge of security arrangement of the user.
- Do not use the appliance into a bathroom or near a water supply source. Do not touch the appliance by wet hands.
- Before putting the appliance in storage it must be completely cooled down.
- Do not use the appliance near water or near water-filled tubs, basins, sinks or other containers.

## OPERATING INSTRUCTIONS

- To connect the nozzles simply insert them in the slot of nozzles.
- To disconnect, pull them out.
- Before disconnecting or connecting the nozzles make sure the speed switch is in the "0" position.
- Set the switch at the desired speed.

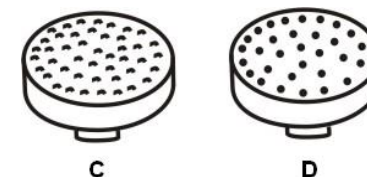
### SAWING AND NAILS SHAPING DISCS

- Discs (A, B) are designed for shaping and sawing thick and thick nails. Use the nozzle that is most suitable for the thickness of your nails.
- Nozzle A - Disk for sawing and nails shape (less rough)
- Nozzle B - Disk for sawing and nails shape (more rough)


### DISCS TO REMOVE ROUGH SKIN

- Discs (C, D) are designed to remove rough skin from the feet and in places where there are rough areas of skin.
- Gently push the said nozzle to areas of rough skin on the feet, heels and treat these places, moving the device forward and back.

### DISC FOR POLISHING

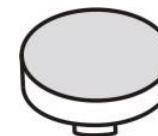- Used to give a smooth and rounded nails after they are processed, and for polishing the surface of the nail at the end of the manicure / pedicure.

### BIG CONE NOZZLE

- Designed for resurfacing of the skin on the soles and heels and remove rough skin around the nail plates on the toes and calluses.

### SMALL CONE NOZZLE

- Designed for fine processing of the nails and removing the skin around the nail plates.

### SMALL CYLINDRICAL NOZZLE

- Designed to remove rough skin around the nail plates and treatment of ingrown nails on the toes. Use, using the movement from the outside of the nail inside.

### **CHOPSTICK AND SPATULA FOR LIFTING/REMOVAL OF CUTICLES**

- Designed for raising the cuticle and its further removal.

### **ROUND BRUSH TO CLEAN NAILS AFTER TREATMENT**

- Designed for cleaning nails after manicure / pedicure or to prepare nails for polishing.

### **RECOMMENDATIONS**

- For the best effect in nail care not to steam arms and legs before the procedure, do it after a manicure / pedicure.
- After the manicure is recommended to immerse the hands in warm, soapy water and then process them in any moisturizer or lotion. This will prevent nails from drying and keep them healthy.

## **CLEAN AND CARE**

- Unplug the appliance from electrical supply network and wait until it completely cooled down.
- Wipe over the housing, using a slightly damp cloth. Do not use any abrasive materials
- Do not allow water or other liquids into the appliance.
- Do not wrap the cord on the body of the device, because this can cause damage to the cord.
- Wipe the nozzles with an alcohol solution.

## **SPECIFICATION**

	Power supply	Net weight / Gross weight	Package size (L x W x H)
LU-2403	220-240 V, 50 Hz Output 3 V, 500 mA	0,54 kg / 0,70 kg	165 mm x 75 mm x 219 mm

### **Producer:**

Cosmos Far View International Limited  
Room 701, 16 apt, Lane 165, Rainbow North Street, Ningbo, China  
Made in China

### **WARRANTY DOES NOT COVER SUPPLIES (filters, ceramic and non-stick coating, rubber seals, etc.)**

Production date is available in the serial number located on the identification sticker on the gift box and/or on sticker on the device. The serial number consists of 13 characters, the 4th and 5th characters indicate the month, the 6th and 7th indicate the year of device production.

Producer may change the complete set, appearance, country of manufacture, warranty and technical characteristics of the model without notice. Please check when purchasing device.

Current information about service centers is available on the website <http://lumme-ru.com>

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